## INSTRUCTIONS FOR USE

**SHEET A CONTENTS**  
 English  
 Français (French)  
 Español (Spanish)  
 Nederlands (Dutch)  
 Deutsch (German)  
 Italiano (Italian)  
 Português (Portuguese)

**SHEET B CONTENTS**  
 Suomi (Finnish)  
 Svenska (Swedish)  
 Norsk (Norwegian)  
 Dansk (Danish)  
 简体中文版 (Simplified Chinese)  
 日本語 (Japanese)  
 繁體中文版 (Traditional Chinese)  
 한국어 (Korean)

18506454 RVE 2011-02 © 2011 Fisher & Paykel Healthcare Limited

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**WARNINGS**

- This mask should be used only with CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly. The exhalation port(s) associated with the mask should never be blocked.
- Explanation of Warning:** CPAP and Bi-Level machines are intended to be used with special masks that have exhalation ports to allow continuous flow of air out of the mask. When the CPAP or Bi-Level machine is turned on and functioning properly, the breath continuity air leak is kept very low via the valve. Likewise the PPC or VNDP is not functioning, the breath continuity air leak will not be reduced, and the exhalation ports will not be provided through the mask and exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can cause suffocation.
- At low CPAP/Bi-Level pressures, the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- Valve activation/deactivation pressure is cm H<sub>2</sub>O.
- Do not use the mask without the Non-Rebreathing Valve in place. This valve ensures CO<sub>2</sub> re-breathing is kept to a minimum in the event of power failure.
- Do not use the mask if it is leaking or leaking excessively.
- Discontinue use of the mask if there is an allergic reaction to any part of the mask. Consult your physician if this occurs.
- If using the FlexiFit™ 432 Full Face Mask between different patients, refer to the Multi-Patient Cleaning Instructions for the FlexiFit™ 432 Full Face Mask.
- Replace the Non-Rebreathing Valve every 6 months.
- Use the FlexiFit™ 432 Full Face Mask Elbow on other Fisher & Paykel Healthcare masks.
- Use the FlexiFit™ 432 Full Face Mask only with the Headgear provided.

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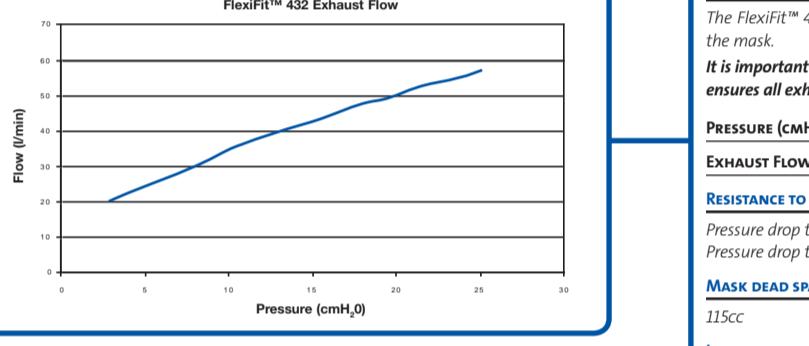
**CONTRAINDICATIONS**

- This mask should not be used by anyone incapable of removing the mask unaided, without continuous supervision.

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The FlexiFit™ 432 Full Face Mask and accessories are latex free. Many people find CPAP therapy causes dryness of the nose and throat. Heated humidification can substantially reduce these symptoms. Contact your supplier and ask about ThermoSmart™.

## TECHNICAL SPECIFICATION

## EXHAUST FLOW INFORMATION

The FlexiFit™ 432 Full Face Mask has exhalation ports to expel the air that you exhale from the mask.  
It is important that these exhalation ports are not blocked by any object. This controlled leak ensures all exhaled CO<sub>2</sub> is sent upwards.

PRESSURE (cmH<sub>2</sub>O)

EXHAUL FLOW (L/MIN)	PRESSURE (cmH <sub>2</sub> O)
21	3
24	5
28	7
31	9
36	11
39	13
41	15
44	17
47	19
50	21
52	23
54	25

## RESISTANCE TO FLOW

Pressure drop through mask @ 50 /min 0.3 ± 0.1 cmH<sub>2</sub>O  
Pressure drop through mask @ 100 /min 1.3 ± 0.1 cmH<sub>2</sub>O

## MASK DEAD SPACE

115cc

## INSPIRATORY AND EXPIRATORY RESISTANCE

The inspiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is 1.2 cmH<sub>2</sub>O.  
The expiratory resistance of the mask (in combination with the Non-Rebreathing Valve) is 2.5 cmH<sub>2</sub>O.

## OPERATING CONDITIONS

Operating pressure range is 3-25 cmH<sub>2</sub>O  
Air temperature of use range is 40-120°F (5-50°C)

## NOISE

The noise level of the mask is 30 dBA, with uncertainty 2.5 dBA.  
The sound pressure level of the mask is 23 dBA, with uncertainty 2.5 dBA.

## SOUND

The sound pressure level of the mask is 30 dBA, with uncertainty 2.5 dBA.

FLEXIFIT™ 432 FULL FACE MASK  
MULTI-PATIENT CLEANING INSTRUCTIONS

Your FlexiFit™ 432 Full Face Mask is provided patient ready. To reuse between patients please follow the guidelines below.

## CLEANING / DESOILING

Before desoiling, disassemble all components from the Mask Base. Clean all surfaces of mask and components thoroughly with a mild detergent solution and a soft bristled brush. Ensure all visible soil is removed. Rinse thoroughly under running potable water to remove detergent residue.

## DO NOT PROPERLY CLEAN THIS DEVICE MAY RESULT IN INADEQUATE DISINFECTION

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THE FLEXIFIT™ 432 FULL FACE MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF DAMAGED.

Disclaimer: It is the user's responsibility to qualify any deviations from the recommended methods of re-processing. Some discolouration may occur after re-processing.

CONTACT YOUR LOCAL HOME CARE PROVIDER FOR ADDITIONAL INFORMATION.

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